




McLaren Photonic Therapy™

The McLaren Method of Treatment © TM

9 Standard Points, plus any pain points

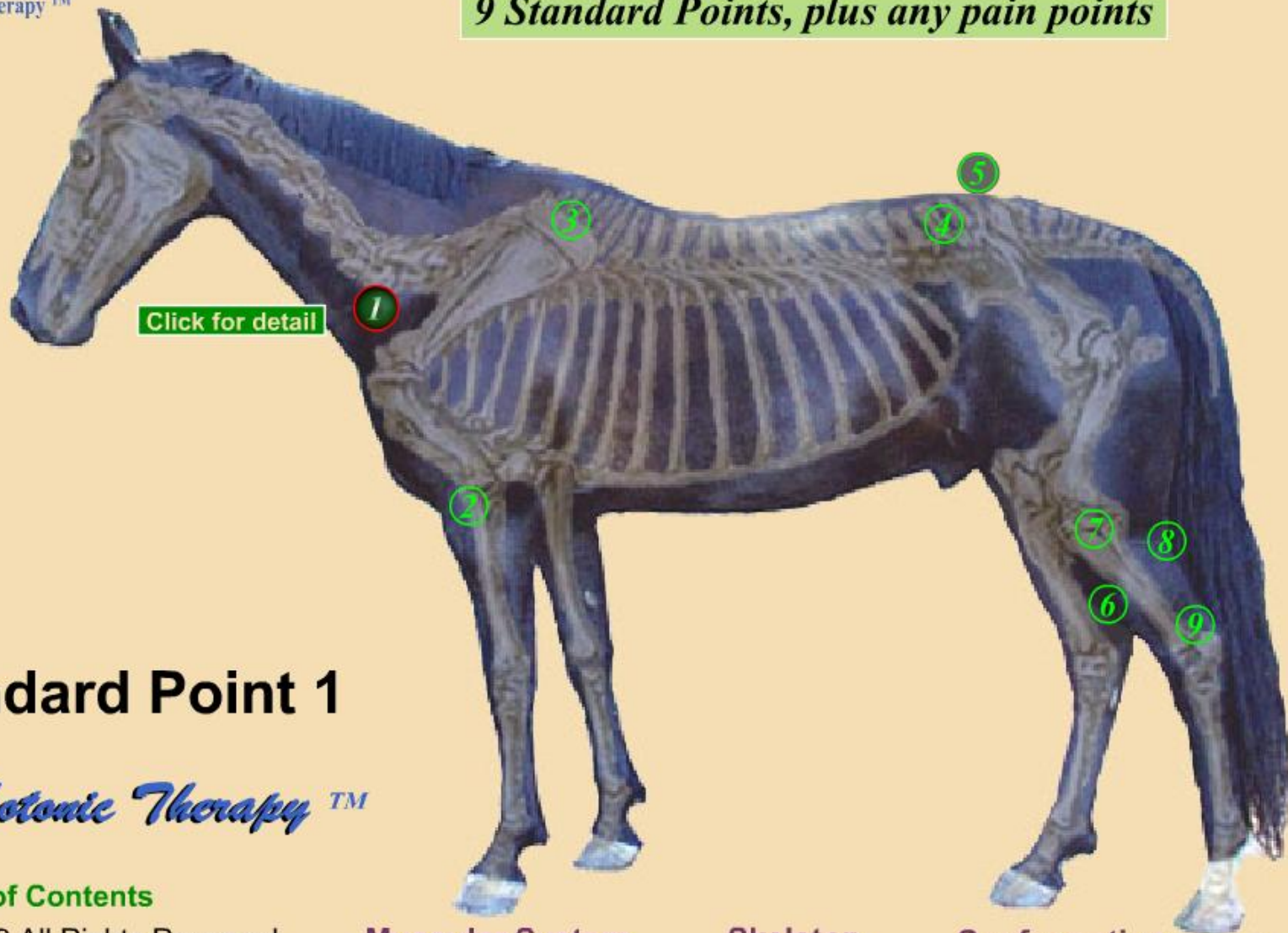

Click for detail

Standard Point 1

Equine Photonic Therapy™


[Table of Contents](#)



McLaren Photonic Therapy™

Location : is located at the base of the neck at the bottom of the "Collar Groove" at the intersection of the Scapular muscle and the brachiocephalicus muscle.

Indication : Reflex point for the opposite lower leg. Rheumatism of the upper fore limb, arthritis of the shoulder and elbow, myositis, paralysis of the nerves of the shoulder and pneumonia.

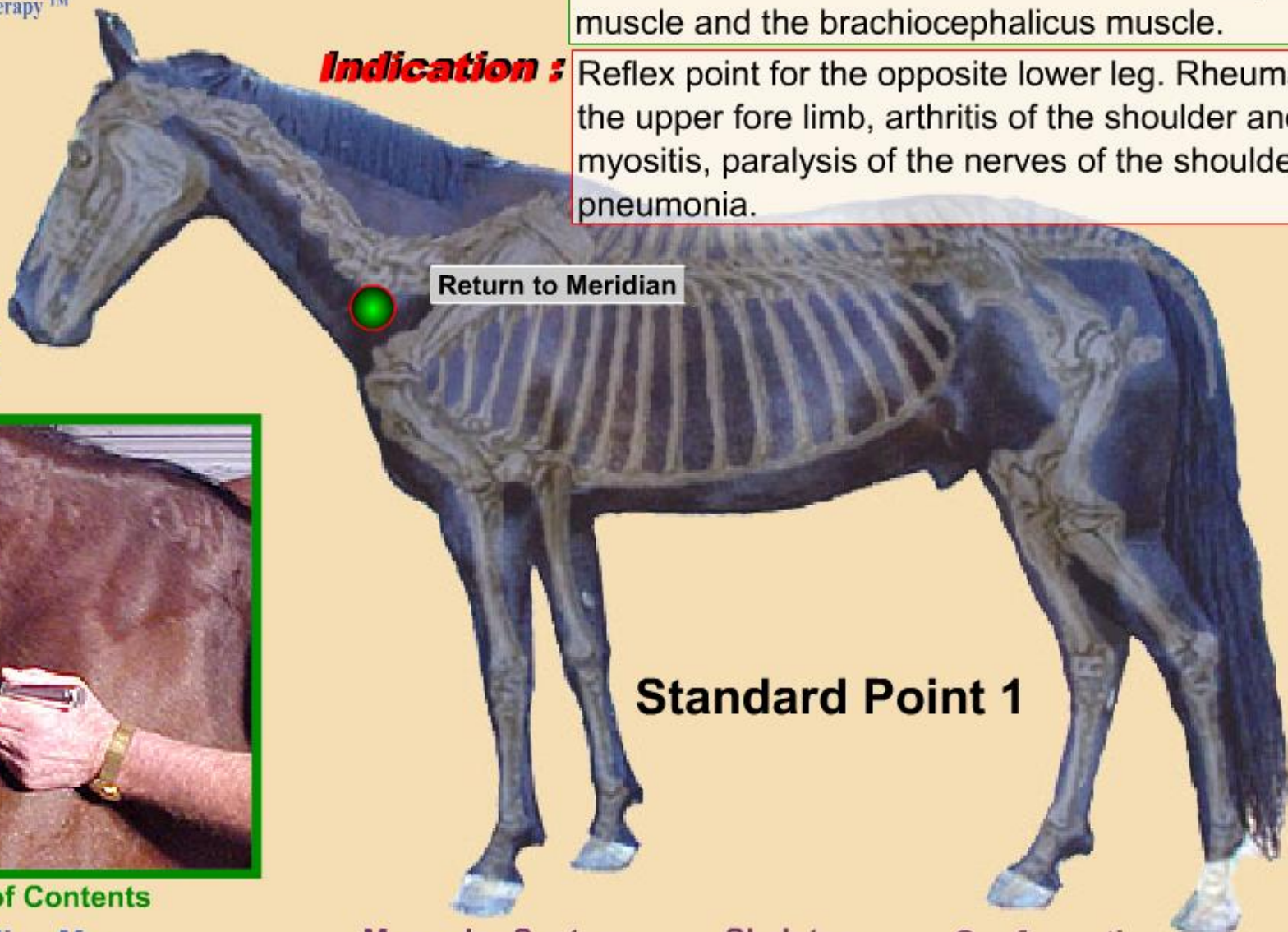

LI 16

Standard Point 1



[Table of Contents](#)

[Standard Meridian Menu](#)

[Muscular System](#)

[Skeleton](#)

[Conformation](#)



McLaren Photonic Therapy™


Location :

is located at the base of the neck at the bottom "Collar Groove" at the intersection of the Scapular muscle and the brachiocephalicus muscle.

Indication :

Reflex point for the opposite lower leg. Rheumatism of the upper fore limb, arthritis of the shoulder and myositis, paralysis of the nerves of the shoulder and pneumonia.

LI 16

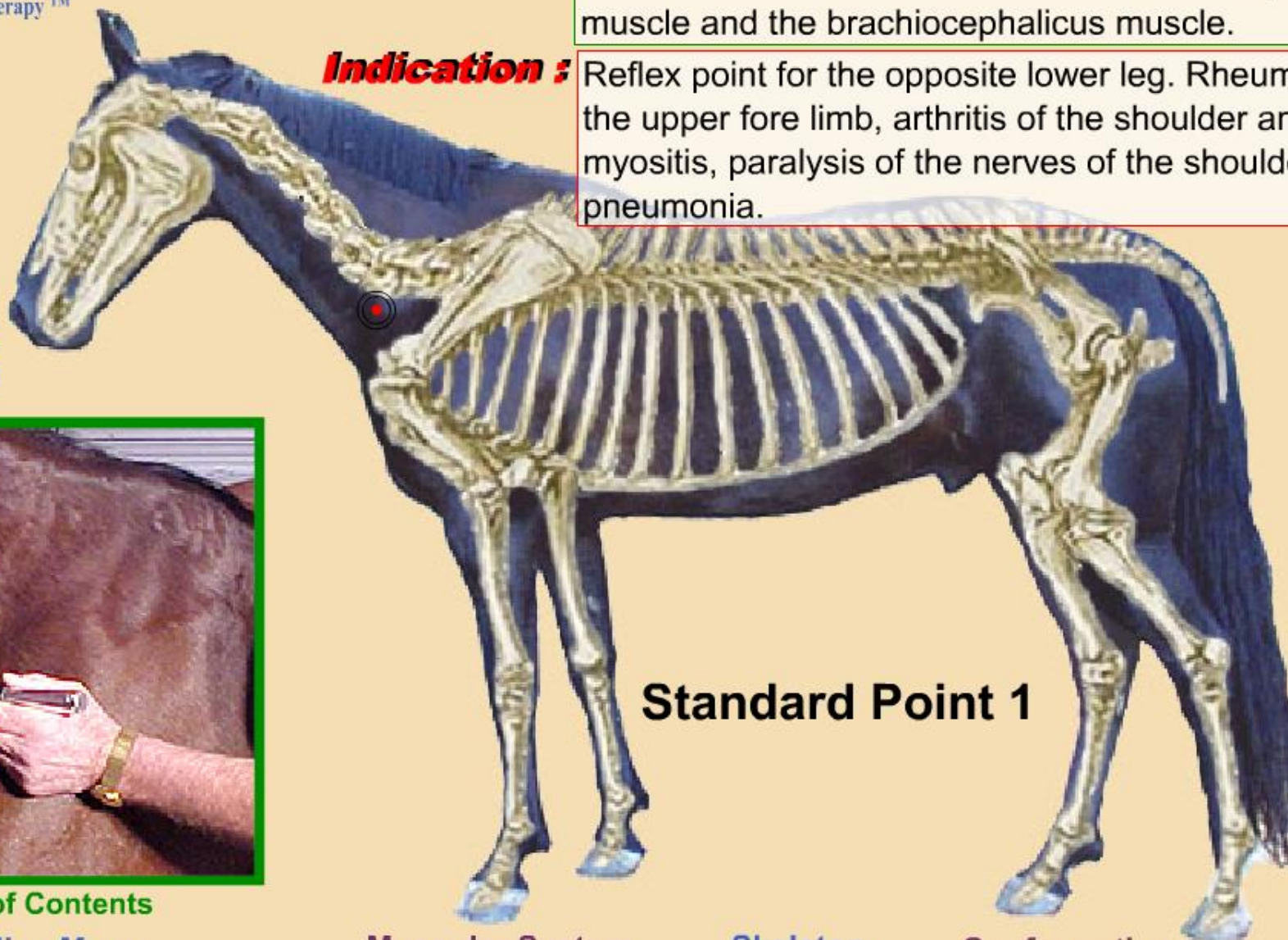

Standard Point 1



[Table of Contents](#)

[Standard Meridian Menu](#)

[Muscular System](#)

[Skeleton](#)

[Conformation](#)

[Fast](#)




McLaren Photonic Therapy™

Location :

is located at the base of the neck at the bottom of the "Collar Groove" at the intersection of the Scapula muscle and the brachiocephalicus muscle.

Indication :

Reflex point for the opposite lower leg. Rheumatism of the upper fore limb, arthritis of the shoulder and elbow, myositis, paralysis of the nerves of the shoulder and pneumonia.

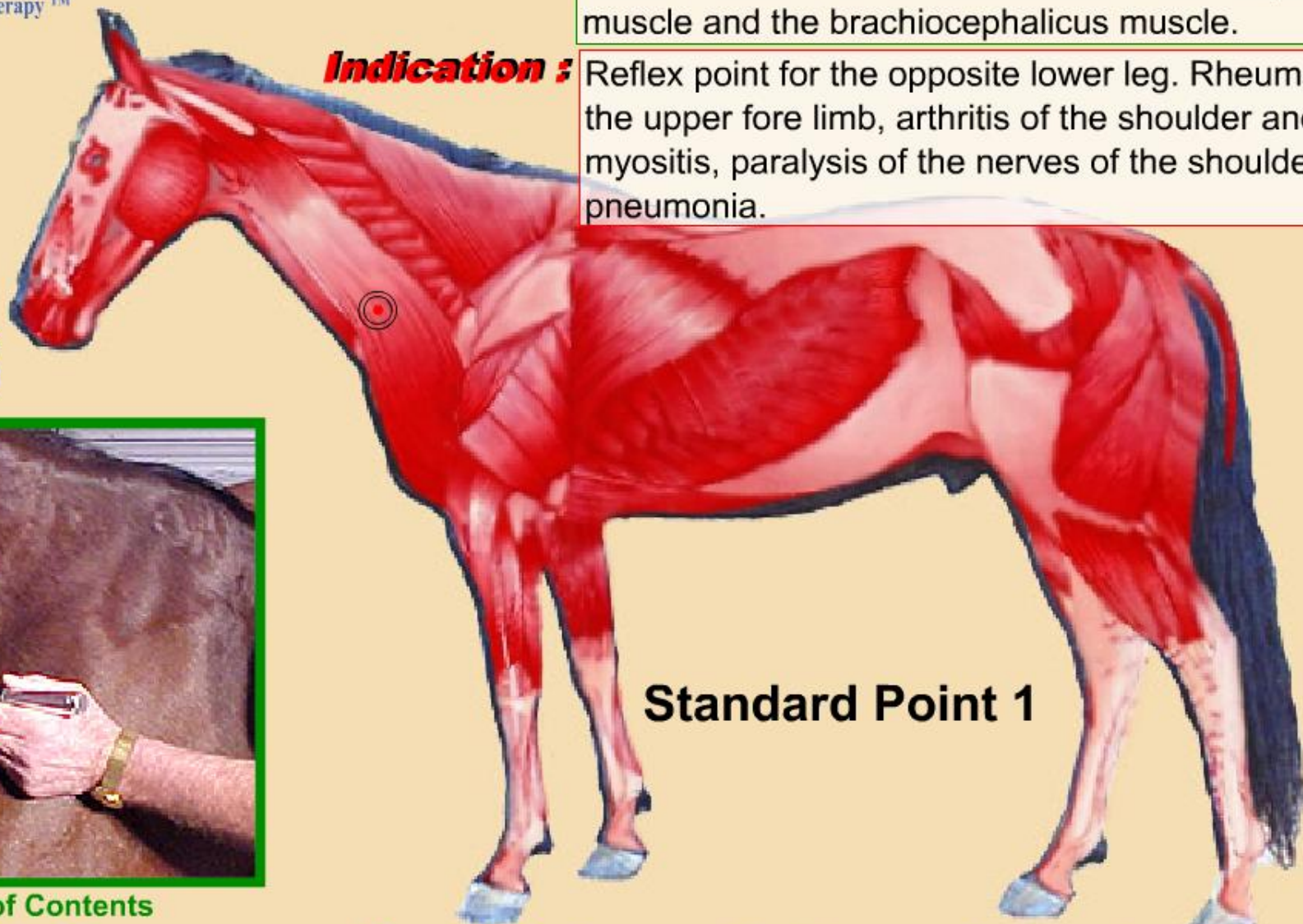

LI 16

Standard Point 1



[Table of Contents](#)

[Standard Meridian Menu](#)

[Muscular System](#)

[Skeleton](#)

[Conformation](#)